



## DATASHEET LIGHT RING OPTION – P/N° 7V2475.01.1



Patented

### PROTECTIVE MEASURES

- WATER RESISTANT
- IMPACT RESISTANT


### RECOMMENDATIONS

- FOR CORRECT USE IT IS REQUIRED A PROPER TRAINING BY A CERTIFIED ORIENTATION & MOBILITY SPECIALIST
- IF YOU HAVE CHOSED TO OPT FOR THE LIGHT RING OPTION AS ADDITIONAL OPTION TO THE AUTOMATIC SWITCHED-ON SYSTEM, THEN A MAGNET HAS PREVIOUSLY BEEN PLACED INSIDE THE BLIND CANE.  
UNDER THIS CIRCUMSTANCES, THE LIGHT RING WORKS CORRECTLY ONLY IF IT IS INSTALLED IN A SPECIFIC WAY; OTHERWISE, IT WOULD NOT WORK PROPERLY IF TOO CLOSE TO THE BLIND CANE'S MAGNET.  
TO IDENTIFY THE CORRECT WAY, YOU JUST HAVE TO PLACE THE LIGHT RING NEAR THE BLIND CANE'S MAGNET:  
-IF THEY ATTRACT EACH OTHER, IT MEANS THIS ONE IS THE RIGHT WAY IT MUST KEEP WHEN THE LAST TWO SEGMENTS ARE PARALLEL (CLOSING POSITION);  
-IF THEY REJECT EACH OTHER, YOU HAVE TO TURN THE LIGHT RING 180 DEGREES CLOCKWISE OR COUNTERCLOCKWISE.

### 1 OFF POSITION

#### Move

the ring towards the tip until it reaches the "magnet label".

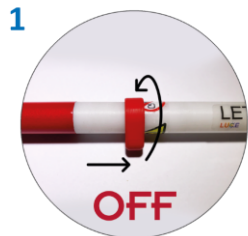


**Rotate** the ring a quarter of round clockwise or counterclockwise keeping its ride until the label's thickness impedes the rotation.



### 2 ON POSITION

#### Move

the ring towards the handle until it reaches the rad band.

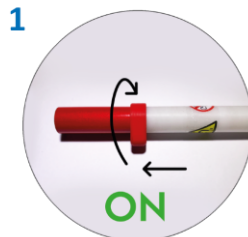

**Rotate** the ring a quarter of round clockwise or counterclockwise keeping its ride until the rad band's thickness impedes the rotation.



### INQUISITIVENESS

- THE LIGHT RING KIT REPRESENTS AN ADDITIONAL OR REPLACEMENT OPTION TO THE AUTOMATIC SWITCHED-ON SYSTEM; IN ORDER TO WORK IT IS NECESSARY THE RESINATION OF A MAGNET INSIDE THE CANE.
- THIS OPTION ALLOWS YOU TO SWITCH THE LETISMART LUCE ON AND OFF MANUALLY JUST SLIDING A RING UP AND DOWN FROM THE OFF POSITION (TOWARDS THE HANDLE) TO THE ON POSITION (TOWARDS THE TIP) AND OTHER WAY ROUND.